



Wall Fastening M2 w/o power

Instruction manual

P/N: 1042100100



CE

Date of issue: 13.05.2025

Rev. A.en

Table of contents

Introduction Safety instructions	4
Intended use	4
Scope of delivery	5
Technical data	5
Accessories	5
Assembly Installation	6
Operation	7
Cleaning Disinfection	8
Symbols used	8
Disposal	9
Service Manufacturer	9

Introduction | Safety instructions

Introduction

Dear Customer,
Thank you for purchasing the Wall Fastening M2 w/o power.

This manual contains technical information and instructions for use and installation.

Safety instructions

- Read the instruction manual carefully before using the Wall Fastening M2 w/o power.
- Keep this manual in a safe place for future reference and pay particular attention to the safety instructions.
- The manufacturer accepts no liability for damage caused by improper operation, use, cleaning, modification or installation.
- Repairs and maintenance on the Wall Fastening M2 w/o power may only be carried out by authorized specialist personnel. In the event of a fault, please contact CRS medical GmbH technical customer service directly. Any modifications to the device or its accessories that are not carried out by the manufacturer will invalidate the warranty and liability obligations of CRS medical GmbH.
- If the EMS ground vehicle was involved in an accident (no minor damage), the bracket must be replaced. Continued use will invalidate the warranty and liability obligations of CRS medical GmbH.
- Check the Wall Fastening M2 w/o power and do not use it if mechanical damage to the locking mechanism is visible.

Intended use

The Wall Fastening M2 w/o power is only intended for the secure mounting of a ZOLL M2® in conjunction with the original ZOLL M2® accessory bag. The use of any other protective case for this device is not permitted.

The Wall Fastening M2 w/o power is designed for in-patient use or for use in an ambulance. The defibrillator used is securely held in the ambulance, allowing the defibrillator to be used on assignment. The bracket may only be used in spatially protected environments. These include solid walls in the interior of buildings and stable walls of EMS ground vehicles.

Please also observe the defibrillator's instructions for use.

Scope of delivery

1 Wall Fastening M2 w/o power (P/N: 1042100100) (UDI-DI: 04251964600195)

1 Instruction manual

Technical data

Description	Data
Weight	3950 g
Dimensions (HxWxD)	362x277x231 mm
Operating temperature	0°C bis +40°C, 5% to 95% RH, non-condensing
Storage temperature	-30°C to +70°C, 5% to 95% RH, non-condensing

We reserve the right to make technical and visual changes to the article in the course of product improvements.

Accessories

No further accessories are available.

Operation

Opening

Unlock the Wall Fastening M2 w/o power by pulling the red handle forwards (1) and then moving it upwards (2). The lock engages in the upper position (3) and the device can be removed.



Closing

Ensure that the ZOLL M2™ with the corresponding accessory bag is properly seated in the lower shell of the Wall Fastening M2 w/o power.

No unlocking is required for closing. Grasp the red handle and push the lid down. The most secure way to lock the Wall Fastening M2 w/o power is to release the cover after overcoming the resistance and allow it to slide into the end position by itself. A clicking sound can be heard.

Cleaning | Disinfection

Please remove the ZOLL M2™ from the holder for cleaning.

The Wall Fastening M2 w/o power can be cleaned with a damp, lint-free cloth and commercially available, mild plastic cleaners. Please ensure that the cleaning agent does not contain any solvents before use.

Also use a commercially available disinfectant for disinfection.

Symbols used



See instruction manual/observe safety instructions



CE marking in accordance with the Medical Device Regulation 2017/745 (Medical Device Regulation)



Reference number



Observe the electronic instruction manual



Unique product identificatio



batch number



manufacturer



Medical device



Recommended temperature range for storage



Recommended relative humidity for storage



Handle the package with care

Disposal

Packaging is a raw material and can therefore be recycled. In the interests of environmental protection, dispose of such in a correctly sorted manner.

Further information on environmentally friendly disposal can be obtained from the manufacturer.

Service | Manufacturer

CRS medical GmbH
Loherstraße 6
D - 35614 Asslar

T: +49 6441 38331-0
F: +49 6441 38331-29
info@crs-medical.com
www.crs-medical.com



Loherstraße 6
D-35614 Aßlar

T: +49 6441 38331-0
F: +49 6441 38331-29

info@crs-medical.com
www.crs-medical.com