Gebrauchsanweisung | User Manual
LP15 Bed Connector
P/N: 1042100044 (vorherige P/N: 2209933003)

■ INHALTSVERZEICHNIS | TABLE OF CONTENTS

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■ WELCOME | SAFETY INSTRUCTIONS

Welcome

Thank you for buying the LP15 Bed Connector.

To ensure safe operation please familiarize yourself with this user manual, please read this manual carefully before use. Keep this manual in a safe place for future reference. We advise you to pay particular attention to the safety instructions.

Safety Instructions

- To ensure safe operation please familiarize yourself with this user manual. Please read this manual carefully before using the LP15 Bed Connector.
- Keep this user manual in a safe place for future reference.
- The manufacturer does not assume liability for damage and/or failure due to incorrect usage or mounting.
- Do not use the LP15 Bed Connector without an inspection.
- Caution: Do not use the connector if you notice:
 - Hook or eyelet is deformed or damaged
 - Safety screw on hook is missing
 - Seam is missing or damaged
 - Zipper, push buttons or Velcro® are not closing properly or are damaged
 - Cover over the handle is torn, damaged or movable

■ APPLICATION

The LP 15 Bed Connector is only for use with a LIFEPAK® 15 monitor/defibrillator made by Physio-Control. Use the connector only on the premises of a hospital to attach the LIFEPAK® 15 monitor/defibrillator on the horizontal frame (head-, foot- or side part) of a hospital bed or an ambulance stretcher.

■ SCOPE OF DELIVERY

1 LP 15 Bed Connector (P/N: 1042100044)

1 User manual

■ TECHNICAL DATA

Weight	120 g
Dimensions (HxWxD)	215x140x65 mm
Distance from handle to frame	approx. 10 cm
Maximal load	11 kg

In the pursuit of product improvement, we reserve the right to make visual and/or technical changes to the product.

■ ACCESSORIES

There are no additional accessories available.

■ INSTALLATION | HANDLING

For proper installation of the LP15 Bed Connector to the handle of the LIFEPAK® 15 monitor/defibrillator:

1. Place the cover over the handle so that the opening of the hooks point towards the rear of the LIFEPAK® 15 monitor/defibrillator.
2. To connect the LIFEPAK® 15 to the Bed Connector correctly, first close the zipper, followed by the Velcro® seal.

3. Make sure that the hanging LIFEPAK® 15 doesn't limit the functionality or usability of the hospital bed and that during use of the hospital bed that the LIFEPAK® 15 doesn't unintentionally unhook, partly or completely, in order to avoid damage to the device.

Observe the following mechanical requirement:

- Max. bedframe tube diameter 3,6 cm
- Use only on horizontal frames

■ CLEANING

The surface of the LP15 Bed Connector can be cleaned with a mild surface cleaner and a moistened cloth.

■ SYMBOLS

See user manual / Safety Instructions

Reference number

Lot number

■ DISPOSAL

In order to protect the environment, please dispose and recycle all materials according to your local laws and regulations. For more detailed information on proper disposal please contact manufacturer.

■ SERVICE | MANUFACTURER

CRS medical GmbH
Loherstraße 6
D - 35614 Asslar

T: +49 6441 38331-0
F: +49 6441 38331-29
info@crs-medical.com
www.crs-medical.com

□ Loherstraße 6
□ D - 35614 Asslar

□ T: +49 6441 38331-0
□ F: +49 6441 38331-29

□ info@crs-medical.com
□ www.crs-medical.com