Gebrauchsanweisung | User Manual
X Series Bed Connector
P/N: 1042100025

■ INHALTSVERZEICHNIS | TABLE OF CONTENTS

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■ WELCOME | SAFETY INSTRUCTIONS

Welcome

Thank you for purchasing the X Series Bed Connector.

In this user manual you will find technical information, instructions for use, and mounting directions.

Safety Instructions

- Read and understand the instructions before installing or operating the X Series Bed Connector.
- Follow all safety instructions and save this user manual in a safe place for future use.
- The manufacturer accepts no liability for damage caused by improper operation, mounting, or assembly.
- Repair and maintenance of the X Series Bed Connector can only be performed by authorized personnel. Please consult directly to CRS medical GmbH Technical Support if the device is defective or damaged. Any changes to the device or its accessories which are not performed by CRS medical GmbH will void all product warranties. CRS medical GmbH will not be liable for unauthorized repair or maintenance.
- Make sure that hanging the ZOLL X Series® does not influence the functionality or usability of the bedside or stretcher and that the defibrillator does not unintentionally unhook during use of the bedside or stretcher in order to avoid damage to the device
- Check the X Series Bed Connector and do not use if:
 - there is visible mechanical damage on the hooks or eyelets
 - the safety screw on one of the hooks is missing
 - zipper or velcro is not closing properly or damaged
 - the cuff around the handle is torn or damaged

■ APPLICATION

The X Series Bed Connector is intended for the secure attachment of a ZOLL X Series® to a bedside or a stretcher during the transfer of patients. The positioning of the product on the bedside or stretcher is provided on horizontal standard rails or round pipes up to Ø36mm.

It is not suitable as a support during transportation in a rescue vehicle. The use of the fastening is only permitted outside EMS vehicles.

Please follow all manufacturer's instructions for the X Series defibrillator.

■ SCOPE OF DELIVERY

1 X Series Bed Connector (P/N: 1042100025)

1 user manual

TECHNICAL DATA

Weight	113 g
Dimensions (HxWxD)	160x125x65 mm
Distance from handle to frame	approx. 10 cm
Maximal load	11 kg

Specifications subject to change without notice.

■ ACCESSORIES

There are no additional accessories available.

■ INSTALLATION | HANDLING

Install the X Series Bed Connector as shown in figure 1 to 3 on the device and turn the cuff afterwards so the eyelets are on top of the handle. Please make sure that the opening of the hooks is pointing to the rear side of the device as shown in figure 4.

Figure 1

Figure 2

Figure 3

Figure 4

■ CLEANING

Please remove the X Series Bed Connector from the device before cleaning.

The X Series Bed Connector can be cleaned with a damp, lint-free cloth and commercially available mild plastic cleaners. Please ensure that the cleanser does not contain any solvents.

Commercially available disinfectants can be used for disinfection.

■ SYMBOLS

See user manual / Safety Instructions

Reference number

Lot number

Manufacturer

■ DISPOSAL

Packing is raw material and therefore recyclable. On behalf of environment protection please dispose the packing sorted. Dispose and recycle all packaging according to your local laws and regulations.

For more detailed information regarding environmentally friendly waste disposal please contact the manufacturer.

■ SERVICE | MANUFACTURER

CRS medical GmbH
Loherstraße 6
D - 35614 Asslar

T: +49 6441 38331-0
F: +49 6441 38331-29
info@crs-medical.com
www.crs-medical.com

□ Loherstraße 6
□ D - 35614 Asslar

□ T: +49 6441 38331-0
□ F: +49 6441 38331-29

□ info@crs-medical.com
□ www.crs-medical.com