Gebrauchsanweisung | User Manual

Bracket AED Pro

P/N: 1042100001 (vorherige P/N: 2100403029)

■ INHALTSVERZEICHNIS | TABLE OF CONTENTS

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■ WELCOME | SAFETY INSTRUCTIONS

Welcome

Thank you for purchasing the Bracket AED Pro.

In this user manual you will find technical information, instructions for use, and mounting directions.

Safety Instructions

- Read and understand the instructions before installing or operating the Bracket AED Pro.
- Follow all safety instructions and save this user manual in a safe place for future use.
- The manufacturer accepts no liability for damage caused by improper operation, mounting, or assembly.
- Repair and maintenance of the Bracket AED Pro can only be performed by authorized personnel. Please consult directly to CRS medical GmbH Technical Support if the device is defective or damaged. Any changes to the device or its accessories which are not performed by CRS medical GmbH will void all product warranties. CRS medical GmbH will not be liable for unauthorized repair or maintenance.
- Check the Bracket AED Pro and do not use if:
 - there is visible mechanical damage on the locking mechanism
 - there is visible mechanical damage on the holder

■ APPLICATION

The Bracket AED Pro is intended to be installed in ambulances which use the ZOLL AED Pro® defibrillator in combination with the softbag listed under accessories. Using a different protective bag is not allowed.

The holder may only be used in a protected environment. These include massive walls in the interior of buildings and stable walls of EMS ground vehicles.

Please also follow all manufacturer's instructions for the defibrillator.

■ SCOPE OF DELIVERY

1 Bracket AED Pro (P/N: 1042100001)

1 User Manual

■ TECHNICAL DATA

Weight	2300 g
Dimensions (HxWxD)	255x335x130 mm
Operating temperature	0°C to +40°C, 5% to 95% HR, non-condensing
Storage temperature	-30°C to +70°C, 5% to 95% HR, non-condensing

Subject to change without notice.

■ ACCESSORIES

The following accessories are not included in the scope of delivery.


Softbag AED Pro (P/N: 1042101009)

AED Pro® with Softbag AED Pro

■ INSTALLATION

Mount the Bracket AED Pro with 4 screws size M6 according to the drilling diagram below.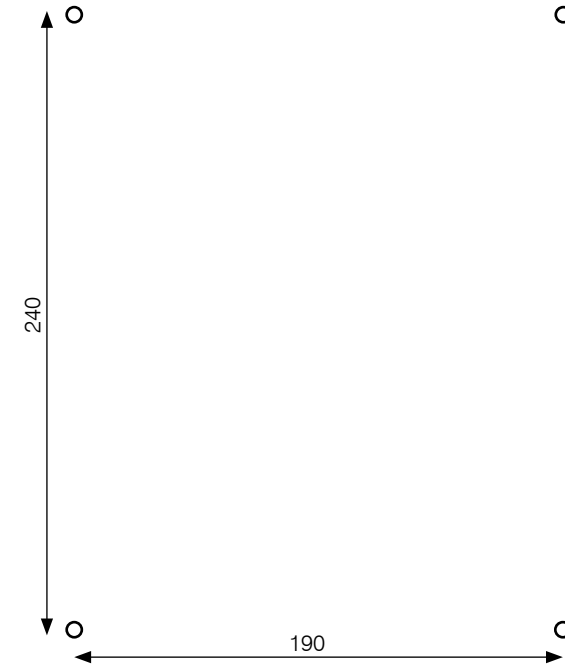

(All dimensions are in mm)

■ OPERATION

Open

Unlock the bracket by pulling the locking bolt and lifting the locking device.

The locking device rests in the upper position so the device can be removed.

Close

Insert the defibrillator and pull the locking bolt to unlock the locking device from the upper position. Press the locking device down until the locking bolt engages hearable in the lower position.

The defibrillator is now secured.

■ CLEANING | DISINFECTION

Please remove the ZOLL AED Pro® out of the bracket before cleaning. The Bracket AED Pro can be cleaned with a damp, lint-free cloth and commercially available mild plastic cleaners. Please ensure, that the cleaner does not contain any solvents. Commercially available disinfectants can be used for disinfection

■ SYMBOLS

See user manual/note safety instructions

Reference

Lot number

Manufacturer

■ DISPOSAL

Packing is raw material and therefore recyclable. On behalf of environment protection please dispose the packing sorted. Dispose and recycle all packaging according to your local laws and regulations.

For more detailed information regarding environmentally friendly waste disposal please contact the manufacturer.

■ SERVICE | MANUFACTURER

CRS medical GmbH
Loherstraße 6
D - 35614 Asslar

T: +49 6441 38331-0
F: +49 6441 38331-29
info@crs-medical.com
www.crs-medical.com

□ Loherstraße 6
□ D - 35614 Asslar

□ T: +49 6441 38331-0
□ F: +49 6441 38331-29

□ info@crs-medical.com
□ www.crs-medical.com